Citra-Lock™

Catheter-lock solution 20 x 5 ml vials

Product Description:
46,7% trisodium citrate in water. Citra-Lock™ is delivered sterile and supplied as a clear colourless solution. The pH is adjusted to approximately 6,7. Aseptic technique must be maintained at all times. Each single-use vial contains 5 ml.

Intended Use:
Citra-Lock™ is intended solely as a catheter lock solution to prevent coagulation of the blood and infection in any type of intravenous catheter.
To prevent:
• Clotting of blood in catheter
• Bacterial infections in catheter
And reduction of:
• Biofilm formation in catheter

Performance:
Trisodium citrate causes anticoagulation by chelation of ionised calcium in the extracorporeal circuit into a soluble complex. Because calcium is an integral ion involved in the clotting cascade, local removal by citrate prevents the activation of clotting cofactors, factor X, and prothrombin, and the ultimate formation of fibrin. Antimicrobial effect and a reduction of biofilm formation through binding and removal of Ca²⁺ in the surrounding milieu. Ca²⁺ may regulate several genes responsible for growth and survival of microbes.

Warnings:
• Citra-Lock™ is intended only for use as Catheter Lock
• Citra-Lock™ may not be applied by either direct intravenous injection or added to an infusion!
• To be used by skilled and trained personnel only!
• Do not use the vial if the seal is broken!
• Citra-Lock™ is for single-use only. Do not use one vial for multiple patients!
• Dispose of unused portion!
• The use of previously opened vials (broken seal) or the use of unused portions may result in bacteraemia or catheter-site infection in the patient due to bacterial contamination of such vials or portions!
• Only inject Citra-Lock™ if the exact catheter volume is known (see instructions from catheter manufacturer!).
• Store at room temperature. Protect from freezing!
• Do not use the product if one is allergic to any of the components!
• Report any serious incident that has occurred in relation to Citra-Lock™ to the manufacturer and to the competent authority having jurisdiction in the Member State in which the user and/or patient is established.

Side-Effects:
• No side-effects known, when the solution is used as catheter lock.
• In the event that Citra-Lock™ unintentionally passes into the vein, paraesthesia and/or dysgeusia may occur.

Instructions for use:
• Aseptic technique must be maintained at all times.
• Prior to initiation of the dialysis session, the Citra-Lock™ that was installed in the catheters during the previous session, should be aspirated with the help of a syringe and discarded according to the institution's biohazards waste policy.
• Flush each catheter lumen with 5 ml of sterile 0,9% sodium chloride solution to remove any blood remaining from the previous operation.
• Extract the exact priming volume (prescribed by the catheter manufacturer) of Citra-Lock™ from the vial, using a 5 ml or smaller graduated syringe, to ensure accurate volume.
• Inject the priming volume of Citra-Lock™ slowly into the catheter (taking 5-10 sec).

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Difinco B.V. Lauwersmeer 9C 5347 JR Oss The Netherlands